The Philips Respironics Detachable Battery Module can be used to power your BiPAP A40 device with a rechargeable Lithium-ion detachable battery. When inserted into the module and connected to the device, the detachable battery will automatically recharge. See the instructions included with the Detachable Battery Pack for more information on the detachable battery.

Cautions:





- Do not open the Detachable Battery Module case. There are no user-serviceable parts inside.
- Do not expose the Detachable Battery Module to water or conductive liquids; this is not a sealed case.
- Use only with the lithium-ion batteries specified below. Other types of batteries may burst causing injury.

Note: For use only with Philips Respironics Lithium-ion batteries (1055804 and 1043570).

Connecting the Detachable Battery Module

To connect the battery module to the device and install the detachable battery, complete the following steps:

1. Remove the side cover on the therapy device. Press in the tab on the side cover and pull the cover away from the device.
2. Line up the Detachable Battery Module and the therapy device side by side, and make sure the guides on the module fit into the slots on the therapy device.
3. Press the two units together until they snap into place. Make sure the therapy device and battery module are completely seated against each other.
4. Insert the detachable battery into the battery module as shown below, until it snaps into place. Close the lid on the battery module.

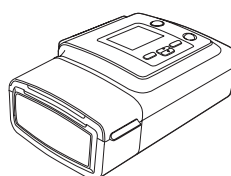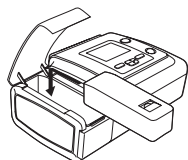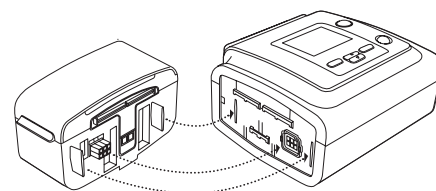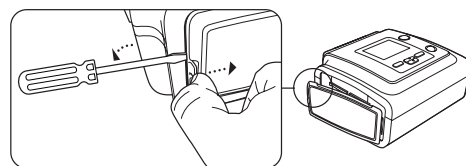Detachable Battery Module Specifications

Specification	Description
Weight	15 ounces
Dimensions	3.8" H x 3.9"W x 6.8" L (9.7 cm H x 9.9 cm W x 17.3 cm L)
Standards Compliance	IEC 60601-1; IEC 60529; EMC: CISPR 11, EN 55022, EN 55024; EN 61000-3-3; IEEE 1625; FCC Part 15

Symbol Key

The following symbols appear on the battery module.

Symbol	Description
	Attention: Consult Accompanying Documents
	Electrical Input
	Electrical Output
	Separate collection for electrical and electronic equipment per EC Directive 2012/19/EU.

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Respironics Inc.
Respironics Deutschland GmbH & Co. KG